35°C

36°C

37°C

38°C

39°C

40°C

LV

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ENG

Maria Forehead Thermometer Strip Your body temperature indicator

Maria Forehead thermometer strip is applied to the forehead to measure the approximate body temperature. Ideal for young children.

Hold the **Maria Forehead** thermometer strip at both ends without touching the temperature indicator and press against a dry, clean forehead for 15 seconds. Read **Maria Forehead** without removing the strip from your forehead. When the colors of the temperature indicator of the thermometer strip have stopped changing, you can read the result. The cell that has changed its color to green will indicate the correct body temperature.

Use the **Maria Forehead** thermometer strip at room temperature.

Do not measure temperature for at least 30 minutes after eating, drinking or exercising, or immediately after coming in from outdoors.

The **Maria Forehead** thermometer strip should be wiped clean with a soft cloth and stored in a box away from heat. Protect from direct sunlight.

ALWAYS CONSULT YOUR DOCTOR IF YOU HAVE AN INCREASED BODY TEMPERATURE

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